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ENG Instructions for use

Edition: AD240524
Catalogue number: DRQR3C

This is a unique pillow that prevents snoring. Its design is intended to provide support to the head and neck while sleeping, to ensure a position that helps reduce blockage of the air passages and enables easier and quieter breathing. The central part of the pillow is lower and thinner than the side parts, so as to ensure a proper sleeping position in which the head and spine are in alignment regardless of the body position adopted during sleep. It is made from memory foam, which adapts to the shape of the body in response to its heat and pressure. The foam slowly returns to its original shape and distributes body weight evenly, ensuring a comfortable night's sleep by providing adequate support for the head, neck, shoulders, and upper back. It prevents snoring and eliminates back pain. The pillow is allergy-free, as the memory foam inhibits the growth of dust mites.

The pillow has a double-sided profiled massaging surface in either a peaked or pleated design, the shape of which further promotes proper air circulation.

The pillow is a medical device manufactured using lightweight and durable materials, developed on the basis of modern design principles. Under the influence of body heat, the memory foam adapts perfectly to the shape of the body, thereby increasing the support surface. This creates excellent conditions for relieving the muscular, ligamentous and joint structures in the cervical spine, which it also stabilises. In addition to its use in physiotherapy and physical therapy, the pillow is a recognised basic means of health intervention for dysfunctions of the cervical spine.

Indications:

- Cervical spine pain relief
- Prevention of muscle tension and discopathy in the cervical region
- Counteracting sleep deprivation leading to cardiovascular diseases, weakened immunity, lowered mood
- Support for the rehabilitation of the spine and muscles in the cervical region
- Maintaining adequate and effective sleep time
- Recommendations for relieving pressure on the cervical vertebrae to maintain the natural curvature of the spine

Contraindications:

- Users with serious cervical spine conditions or permanent lesions should consult a physician or physiotherapist before first use.

Maintenance: The memory foam is not washable. The outer pillowcase is removable and can be washed at up to 30°C/ 86°F. The pillow may have a 'factory smell' when removed from its packaging due to the method of transport. To make it disappear, simply air the pillow before first use.

Composition:

Pillow: 100% PU
Outer pillowcase: 100% polyester
Inner pillowcase: 100% polyester
Size: 57x36x11.5 cm
Density: 55 kg/m³ (soft)

Intended user:

The medical device is intended for the general public (a non-professional user). Disposal of the medical device: The device must be disposed of following the applicable local environmental regulations.

Caution: Any serious incident that occurs in connection with a device must be reported to the manufacturer and the competent authority of the Member State (applicable to the EU) where the user and/or patient is based.

Description of symbols used:

 Manufacturer	 Lot/batch number	 Medical device
 Date of manufacture	 Unique identification code of the product-type	 Compliant with Regulation (EU) 2017/745
 Warning	 Unique catalogue number	 Refer to the instructions for use

 **mdh Sp. z o.o.**
ul. Maratońska 104
94-007 Łódź, Polska (EU)
tel. +48 42 674 83 33
e-mail: port@meyragroup.com

ANTI SNORING PILLOW ALMOHADA ANTIRRONQUIDOS PARA DORMIR ANTI SNORING PILLOW REHABILITATIONSKISSEN FÜR DEN SCHLAF

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